

Quarterly Promotions!



Intra-oral Adjustment/Polishing
for Zirconia and Lithium Disilicate

Adjust and Polish Zirconia and
Lithium Disilicate the **eZr** Way!



Intra-oral **Adjustment:**



- **No chatter, chipping or microfracture**
– Unlike diamonds bonded directly to a hard metal shaft, eZr's rubber substrate cushions the diamonds to prevent damage to the restoration.
- **No heat build-up** – The carefully engineered substrate does not transfer heat during use.
- **FAST!** – eZr's coarse grit blue grinders make your adjustments fast and convenient.

Kit	Qty.	Price
FPZK01 Complete system kit	2 grinders 6 polishers	\$311

Canadian MSRP



1 Start by gross adjusting and contouring using the blue coarse grit grinders. Use the flat end taper for occlusal/labial/lingual/buccal surfaces and the disc for interproximal surfaces.



2 Use the green medium grit polishers for smoothing and pre-polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



3 Use the orange fine grit polishers for high gloss polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



Beautiful
results!



Intra-oral **Polishing:**



- **Unparalleled smooth polish**
– Don't take our word for it! – "The eZr system is the only one I've tried that polishes zirconia to smooth!" Todd Brower, DDS, Holland, Michigan.
- **Increased physical stability** – Highly polished zirconia and lithium disilicate is stable and non-abrasive to opposing dentition.
- **No paste required** – Forget the mess and expense of polishing pastes. eZr is all you need.
- **No water required** – No need to use water when polishing. Water is only required for high speed.

Slow speed right angle latch shank
Steam autoclaveable



Buy a eZr kit (FPZK01)
Get a fine grit small point polisher
(FPZF010) 5 pack
> FREE! <

\$77 CAD value

*To redeem goods, email a scanned copy of your dealers invoice dated 1/10/25 - 31/12/25 to: Garrison Dental Solutions, Attn: Liz Kammers to ekammers@garrisdental.com with INCA25Q4 in the Subject line. Allow 4-6 weeks for delivery.

Garrison

Toll-free 888.437.0032
gds@garrisdental.com
www.garrisdental.com

ORDER TODAY! 888.437.0032
or contact your Dealer Representative.

INCA25Q4

© 2025 Garrison Dental Solutions, LLC