




# Valuemed Professional Opti-Kleen System™

Cleaning for delicate micro instruments

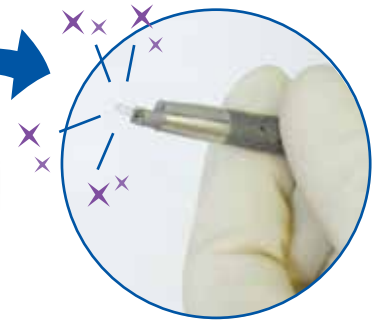

See below for a special **SPRING PROMOTION**


## Efficient, inexpensive, simple, and safe micro instrument cleaning

Opti-Kleen Cleaning System cleans tissue and other debris from the surfaces of delicate micro instruments like those used in cataract and other precision surgical procedures prior to sterilization. By utilizing a proprietary soft water soaked foam sponge material that loosens proteins and other deposits, micro instruments are cleaned safely & pristinely without scratching, dulling, chipping, or breaking.

## Clean pristine micro instruments, as easy as 1, 2, 3...


Cleaning takes less than 15 seconds - Simply clean, rinse, and rinse again

### How it works

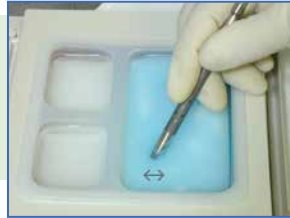


1

Insert the soiled micro instrument into the bottom blue colored soft cleaning pad. Make several passes in a cutting-like fashion to allow the gentle scrubbing to remove the soil from the surface and deposit it into the pad.

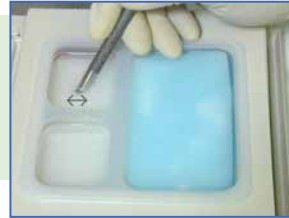

2

Follow this procedure again in the upper right white colored rinse pad.

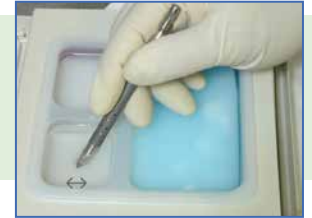

3

Repeat the procedure in the upper left white colored rinse pad for a final rinse.

Rinse pad steps may be reversed if the user prefers, as long as the bottom blue cleaning pad is used first.



## SPRING PROMO

Buy 2 boxes of trays and get 1 cradle **FREE!**

Order No.	Ordering Information:	Price:
VOKD-001	12 trays per dispenser box	\$145.00
VOKHT-001	Cradle (tray holder)	\$ 74.99

Spring promotion valid until May 15, 2021. Limit of 4 cradles per customer account.



**Valuemed Professional Products Ltd.**

1-800-919-8444 | sales@valuemed.ca | www.valuemed.ca

