



16410-117 Avenue,  
Edmonton, Alberta, Canada, T5M 3W2  
**Toll Free Phone: 1 (800) 919-8444**  
**Toll Free Fax: 1 (844) 454-6420**  
**Email: [sales@valuemed.ca](mailto:sales@valuemed.ca)**  
**Website: [www.valuemed.ca](http://www.valuemed.ca)**

## Reusable Instrument Cleaning Brushes MIFU

Product Codes 913-28-72550 / 913-29-72550

### DESCRIPTION/INTENDED USE

Valuemed's reusable brushes are intended to assist in cleaning of medical devices in conjunction with a suitable cleaning solution. All components used in the manufacturing of our reusable brushes are made of commonly used materials and do not require special sterilization handling outside your facility's standard sterilization protocol. At your discretion, please feel free to consult with your sterilization equipment supplier as they should have a list of acceptable serializable components/materials.

### WARNING/PRECAUTIONS

- Consult the manufacturer's IFU for the device you are cleaning and follow those instructions closely.

### Pay close attention to recommendations on the material composition of the brush trim (nylon, stainless-steel).

- Brushes are intended for use as cleaning tools only. They are not for use in patient care or aseptic techniques.
- Avoid brush compression during sterilization and storage.

### CONTRAINDICATIONS

Do not use these brushes for any purpose other than their intended use. Patient injury, cross-contamination and equipment damage may occur.

### INSTRUCTIONS FOR USE

1. Pick the correct type of cleaning brush for the cleaning process.
2. Between usages, inspect for wear, fraying or damaged bristles. If observed, the brush should be discarded in compliance with your facility disposal instructions.
3. *Cleaning and Sterilization:* These products have been manufactured using materials that withstand repeated steam sterilization cycles achieving 134 °C (273 °F). Although the product is designed for considerable re-use, variables such as frequency of sterilization, cleaning application and handling can impact the product's useful life. When cleaning the brush ensure it is free from all foreign material and thoroughly cleaned before being processed at sterilization temperatures. Following sterilization or any established cleaning protocol, examine the brush for any loose bristles, overly worn bristles or cracks on the sheaths.

### STORAGE

Store in a manner that will reduce cross-contamination. Keep in cool dry place and away from direct sunlight.

### DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of the product in accordance to facility protocol with accepted medical practice.