

DIAFIL™ **Bulk FLOW**

Universal Shade

Light Cured Single Shade Bulk Fill Flowable Resin



THE TRIFECTA

ONE SHADE —

Universal shade blends naturally in posterior restorations, reducing decision fatigue & simplifying inventory.

ONE CURE —

Reliable deep-cure performance supports efficient bulk-fill up to 5mm.

ONE STEP —

Smooth, controlled flowability ensures excellent cavity adaptation & minimizing voids.

DiaDent®

KEY FEATURES & BENEFITS

Efficient Bulk-Fill Placement

Save valuable chair time. DiaFil Bulk Flow delivers fast, single-increment placement up to a full 5 mm depth, achieving complete polymerization and exceptional wear resistance – eliminating the need for an additional capping layer.

Strength & Wear Resistance

Engineered with high flexural strength (123 MPa) and high compressive strength (284 MPa) to confidently support deep posterior restorations.

Excellent Flow & Adaptation

Formulated with optimal viscosity and flowability to wet cavity walls perfectly, eliminating gaps and marginal leakage. Glides smoothly into tight margins and difficult-to-access preps without slumping or running.

High Radiopacity

Clear radiographic visibility supports easy post-operative evaluation and confident diagnosis.

Right Shade - First Try



With incredible shade-matching properties, DiaFil Bulk Flow's single universal shade simplifies your workflow, eliminates shade-matching steps, and reduces inventory clutter.

INDICATIONS

- Base under class I and II direct restorations
- Liner under direct restorative materials
- Repair of composite/ceramic veneers
- Restorations of posterior cavities
- Undercut block-out
- Pit and fissure sealant



PROVEN CLINICAL PERFORMANCE

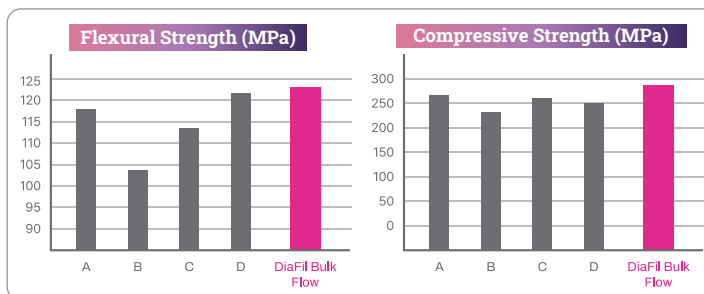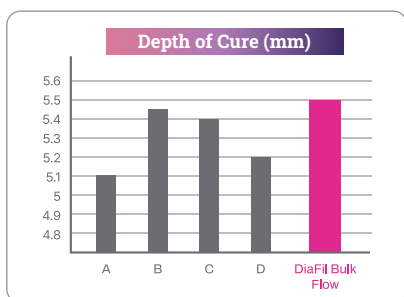

Brand*	Depth of Cure (mm)	
	Mean	SD
A	5.10	0.21
B	5.45	0.06
C	5.40	0.26
D	5.20	0.13
DiaFil Bulk Flow	5.51	0.14

Brand*	Flexural Strength (MPa)		Compressive Strength (MPa)	
	Mean	SD	Mean	SD
A	118	5.33	266.86	12.79
B	103.74	13.22	230.19	2.49
C	113.17	12.03	256.59	18.48
D	121.28	2.65	250.98	8.11
DiaFil Bulk Flow	123.44	1.22	284.12	3.19

Source: DiaDent R&D. Data on file.

*A: Charisma®, B: Tetric® N-Flow, C: DenFil™, D: Beautifil®-Bulk

ORDER INFORMATION



ITEM # PRODUCT INFO

A2004-6909	Econo Pack (5 x 2g syringes + 25 tips)
A2004-6910	Refill pack (1 x 2g syringe + 5 tips)
A2004-6120	Dispensing Tips (20) 18 gauge



For more info &
FREE sample:



DiaDent®

DiaDent Group International
#380 - 8210 Ross Street
Vancouver, BC V5X 0L1

Phone: 1-877-342-3368
Web: diadent.com
Email: diadent@diadent.com