



» I really like Luna 2. It handles great in my hands and is super aesthetic! «

DR ONA ERDT - "THE DENTAL ADVISOR" CLINICAL EDITOR - UNITED STATES


CELEBRATING THE 50TH ANNIVERSARY OF SDI

LUNA 2

Luna 2 is the next generation Luna universal composite. Luna 2 has an improved unique blend of fillers that were researched and developed at SDI to deliver enhanced handling, very high radiopacity and reliable mechanical properties. Luna 2 is BPA free for health-conscious patients. Luna 2 makes placing a composite easier than ever.



👫 Images courtesy of Dr Neus Sancho Lecina - Spain

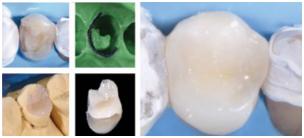

Indirect onlay with Luna 2. Images courtesy of Dr Giuseppe Lacona - Italy





📀 Images courtesy of Prof Dr Marco Aurélio Chaves da Silva - Brazil





👯 Images courtesy of Dr Michael Chan - Australia



 "Handling is fantastic - much improved. It is SDI's best handling composite yet."
DR MICHAEL CHAN - AUSTRALIA



"For a new product to be accepted and incorporated into the dental practice, it must solve a problem (like simplifying shade selection) and be user friendly.
You have a winner in LUNA 2!"
DR FAY GOLDSTEP - CANADA



"LUNA 2 is an extremely versatile composite, with excellent viscosity that ensures efficient handling and success in primary anatomy." <</p>

📀 PROF DR LEONARDO UBALDO - BRAZIL

 "LUNA 2 ticks all the boxes for me.
It's easy to handle, it blends seamlessly and polishes to a nice shine."
DR BILL GERGIS - AUSTRALIA



 "LUNA 2 is a great simplified and aesthetic approach to both anterior and posterior restorations."
DR SAM KOH - AUSTRALIA



"LUNA 2 is certainly the material that I want in my clinical routine and recommend to my students."
PROF DR CRISTIANE DE MELO ALENCAR - BRAZIL




"LUNA 2 is spectacular!" DR GIUSEPPE IACONA - ITALY



FASTER, MORE ACCURATE SHADE MATCHING

Luna 2 offers innovative Logical Shade Matching Technology and the Chameleon Effect for easier and more precise shade selection. Shades were developed prioritising value (lightness), while pigments were arranged in a logically increasing way, for a simpler and faster match to the Vita Classical Shade Guide[®]. Patients will enjoy more accurate aesthetic results, including polishing and wear resistance for long lasting restorations.

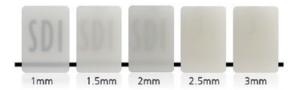



the QR code for free

and for more Luna 2 content.

download of the complete Luna 2 Translucency Clinical Guide

Translucency in different thicknesses of Luna 2 shade A2





O Images courtesy of Dr Jiovanne Neri - Brazil

💻 DR SUSAN MCMAHON - UNITED STATES



Most mastication forces are compressive in nature, making this an important mechanical property for posterior restorations to support the occlusal load without fracturing.



Luna 2 withstands masticatory forces in a similar way to dentine. This reduces failure risk either in areas of high stress, as Class V, or in less supported areas, such as cusp replacement or proximal boxes.



NATURAL TOOTH-LIKE PROPERTIES ACROSS ALL LIGHT


Luna 2 technology mimics the natural properties of teeth across all light sources. Your patients will enjoy lifelike translucency, opalescence, and fluorescence, with outstanding clinical performance.

PERFECT CREAMY HANDLING

Luna's handling has been further improved. Luna 2 has non-stick, nonslump handling. Dentists conducting clinical trials of Luna 2 have praised its outstanding handling and ease of packing and sculpting.



With dentine/enamel at 100/170% Al radiopacity, Luna 2's outstanding radiopacity allows clear differentiation, providing easy and precise visualisation for diagnosis assistance.

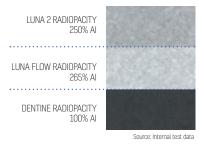




Luna 2 is formulated without BPA derivatives, perfect for health-conscious patients.



The Luna 2 filler formulation is primed for excellent polishing results, wear resistance and longevity.



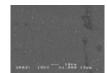

SEM before finishing and polishing

SEM after finishing and polishing

SOURCE: Independently tested by Unité de Recherches Biomatériaux Innovants et Interfaces (URB2I-EA 4462 Paris Descartes). Chairman: Dr Jean-Pierre Attal

INSTRUCTIONS



Etch tooth surface with Super Etch 37% phosphoric acid for 10 seconds

ORDER DETAILS

LUNA 2

LUNA 2 INTRO KIT

Zipbond Universal

Luna 2 Intro Kit with 5 syringes &

5 x 4g Syringes (1 each - A1, A2, A3, A3.5, B1) 2 x 2mL Super Etch Gel Syringes

25 x Super Etch Disposable Tips

LUNA 2 SYRINGE 4G REFILLS

Luna 2 Syr A1 - Vita® A1

Luna 2 Syr A2 - Vita® A2

Luna 2 Syr A3 - Vita® A3

Luna 2 Syr A3.5 - Vita® A3.5

Luna 2 Syr B1 - Vita® B1

Luna 2 Syr B2 - Vita® B2

Luna 2 Syr C2 - Vita® C2

Luna 2 Syr Opaque OA2

opaquer with A2 shade Luna 2 Syr Opaque OA3

opaquer with A3 shade

equivalent to OM2 Vita®

Luna 2 Syr I - Incisal: more

translucent, equivalent to enamel Luna 2 Syr XB - Extra Bleach shade:

Luna 2 Syr 2XB - 2x Extra Bleach

shade: equivalent to OM1 Vita®

ELUNA 2

1 x 5mL Zipbond Universal

40 x Points, fine tip (white)

2 x mixing well - dual

Wash thoroughly, (2) and keep moist

8452025

8452001

8452002

8452003

8452004

8452006

8452007

8452010

8452015

8452016

8452018

8452021

8452022

remove excess water

Apply Zipbond Universal Adhesive (SDI Limited) according to instructions, blow gently for 5 seconds and light-cure for 10 seconds



Place Luna 2 in increments of 2mm or less. Light-cure each layer for 40 seconds (opaque shades) or for 20 seconds (remaining shades). Finish and polish with Polishing Paste (SDI Limited)





LUNA 2

LUNA 2 COMPLET REFILLS

Luna 2 Comp A1 - Vita® A1

Luna 2 Comp A2 - Vita® A2

Luna 2 Comp A3 - Vita® A3

Luna 2 Comp B1 - Vita® B1

Luna 2 Comp B2 - Vita® B2

Luna 2 Comp C2 - Vita® C2

Luna 2 Comp Opaque OA2 ·

Luna 2 Comp Opaque OA3 -

Luna 2 Comp I - Incisal: more

shade: equivalent to OM1 Vita®

translucent, equivalent to enamel

Luna 2 Comp XB - Extra Bleach shade:

Luna 2 Comp 2XB - 2x Extra Bleach

opaquer with A2 shade

opaquer with A3 Shade

equivalent to OM2 Vita®

Luna 2 Comp A3.5 - Vita® A3.5

(0.25G X 20 REFILLS; 5G TOTAL)

(3)





8452051

8452052

8452053

8452054

8452056

8452057

8452060

8452065

8452066

8452068

8452071

8452072



Premium Applicator

APPLICATORS

Premium Complet Applicator

5545022

CLINICAL TIP: XB and 2XB Bleach shades are excellent for use in restorations after tooth whitening treatment with Pola (SDI Limited), and also on milky white teeth.



ADVANCED TOOTH WHITENING SYSTEM

CLINICAL TIP: Luna 2 can be used with Zipbond Universal (SDI Limited) - a single component fluoride-releasing universal adhesive, with excellent handling and desensitizing effect



VIEW A LUNA 2 RESTORATION

Scan the OR code below



YOUR **our** Smile **Vision.**

MADE IN AUSTRALIA by SDI Limited Bayswater, Victoria 3153 Australia www.sdi.com.au

AUSTRALIA 1800 337 003 AUSTRIA 00800 0225 5734 BRAZIL 0800 770 1735 FRANCE 00800 0225 5734 GERMANY 0800 100 5759



ITALY 00800 0225 5734 NEW ZEALAND 0800 808 855 **SPAIN** 00800 0225 5734 UNITED KINGDOM 00800 0225 5734 USA & CANADA 1 800 228 5166



LUNA 2 CURING TIME 40 sec - opaque shades 20 sec - remaining shades

LUNA 2 DEPTH OF CURE



1