



Corporate Office  
1720 Sublette Avenue  
St. Louis, MO 63110

September 9, 2022

Circumcision Clamp Autoclave Process for Reference Only:

1. Dis-Assemble the clamp so all surfaces are exposed.
2. Wash the clamp with a cleaning solution to remove any foreign material.
3. Pre-Vacuum Steam, 4 min. @ 132°C/270°F with 20 min. Dry Time
4. Gravity Displacement Steam, 15 min. @ 132°C/270°F with 15 – 45 min. Dry Time
5. Make sure autoclave chambers are cleaned regularly as recommended by the manufacturer.

This is a process used by others. Verification should be done at the users facility with their own equipment. This is to be used only a guide.

Respectfully,

A handwritten signature in blue ink, appearing to read "Craig Petzon", is written over the word "Respectfully,".

Craig Petzon

Manager, Technical Support.